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## COBAS MIRA S / PLUS

BIOQUIMICA

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	20.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	T1
DECIMAL POSITION	2	LAST	18
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	5 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	30 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	250 µL	STD-2	
		STD-3	
CALCULATION		REPLICATE	DUPL
SAMPLE LIMIT	NO	DEVIATION	10.0 %
REACTION DIRECTION	INCREASE	CONTROL	
CHECK	ON	CS1 POS	*
CONVERSION FACTOR	1.000	CS2 POS	*
OFFSET	0.000	CS3 POS	*

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GENERAL		TEST RANGE LOW	0.00 g/dL
MEASUREMENT MODE	ABSORB	HIGH	6.00 g/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	600 nm	READING FIRST	CB
DECIMAL POSITION	2	LAST	3
UNIT	g/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	NO	TIME	
CONC. FACTOR	NO	BLANK RANGE LOW	0.0000 A
SAMPLE CYCLE	1	HIGH	0.1500 A
VOLUME	2.0 µL	REAGENT RANGE LOW	-0.1000 ΔA
DILUTION NAME	H2O	HIGH	0.2000 ΔA
VOLUME	30.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	300 µL	STD-2	NO
CALCULATION		STD-3	NO
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	OFF	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	NO
MEASUREMENT MODE	ABSORB	HIGH	1000
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	KINETIC
WAVELENGTH	405 nm	READING FIRST	5
DECIMAL POSITION	1	LAST	13
UNIT	U/L	REACTION LIMIT	NO
		CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	2.5 $\mu$ L	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10.0 $\mu$ L	FACTOR:	8209
REAGENT CYCLE	1	STANDARD POS	*
VOLUME	250 $\mu$ L	CONTROL	
CALCULATION		CS1 POS	*
SAMPLE LIMIT	NO	CS2 POS	*
REACTION DIRECTION	INCREASE	CS3 POS	*
CHECK	ON		
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		TEST RANGE LOW	NO
MEASUREMENT MODE	ABSORB	HIGH	350
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	KINSEARCH
WAVELENGTH	340 nm	READING FIRST	3
DECIMAL POSITION	1	LAST	11
UNIT	U/L	REACTION LIMIT	NO
		CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	8	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	15.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10.0 µL	FACTOR:	3086
REAGENT CYCLE	1	STANDARD POS	*
VOLUME	150 µL	CONTROL	
CALCULATION		CS1 POS	*
SAMPLE LIMIT	NO	CS2 POS	*
REACTION DIRECTION	DECREASE	CS3 POS	*
CHECK	ON		
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		TEST RANGE LOW	NO
MEASUREMENT MODE	ABSORB	HIGH	440
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	KINSEARCH
WAVELENGTH	340 nm	READING FIRST	3
DECIMAL POSITION	1	LAST	11
UNIT	U/L	REACTION LIMIT	NO
		CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	8	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	15.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10.0 µL	FACTOR:	3086
REAGENT CYCLE	1	STANDARD POS	*
VOLUME	150 µL	CONTROL	
CALCULATION		CS1 POS	*
SAMPLE LIMIT	NO	CS2 POS	*
REACTION DIRECTION	DECREASE	CS3 POS	*
CHECK	ON		
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		TEST RANGE LOW	0.00 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	15.00 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLP AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	550 nm	READING FIRST	2
DECIMAL POSITION	2	LAST	14
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	NO	TIME	
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	3	HIGH	NO
VOLUME	15.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	20.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	250 µL	STD-2	-
CALCULATION		STD-3	-
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	*
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.00 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	15.00 mg/dL
REACTION MODE	R-S-SR1	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	550 nm	READING FIRST	1
DECIMAL POSITION	2	LAST	10
UNIT	mg/dL		
ANALYSIS		CALIBRATION	
POST DIL. FACTOR	NO	CALIB. INTERVAL	ON REQUEST
CONC. FACTOR	NO	TIME	
SAMPLE CYCLE	2	BLANK RANGE LOW	NO
VOLUME	25 µL	HIGH	NO
DILUTION NAME	H2O	REAGENT RANGE LOW	NO
VOLUME	10.0 µL	HIGH	NO
REAGENT CYCLE	1	STANDARD POS	*
VOLUME	375 µL	STD-1	@
START R1 CYCLE	1	STD-2	-
VOLUME	15 µL	STD-3	-
DILUENT NAME	H2O		
VOLUME	10.0 MI	REPLICATE	DUPL
CALCULATION		DEVIATION	*
SAMPLE LIMIT	NO		
REACTION DIRECTION	INCREASE	CONTROL	
CHECK	ON	CS1 POS	*
CONVERSION FACTOR	1.000	CS2 POS	*
OFFSET	0.000	CS3 POS	*

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GENERAL		TEST RANGE LOW	NO
MEASUREMENT MODE	ABSORB	HIGH	20 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	600 nm	READING FIRST	1
DECIMAL POSITION	1	LAST	6
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	NO	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	2	HIGH	NO
VOLUME	3.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	300 µL	STD-2	NO
		STD-3	NO
CALCULATION		REPLICATE	DUPL
SAMPLE LIMIT	NO	DEVIATION	10.0 %
REACTION DIRECTION	INCREASE	CONTROL	
CHECK	ON	CS1 POS	*
CONVERSION FACTOR	1.000	CS2 POS	*
OFFSET	0.000	CS3 POS	*

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GENERAL		TEST RANGE LOW	NO
MEASUREMENT MODE	ABSORB	HIGH	20 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	550 nm	READING FIRST	1
DECIMAL POSITION	2	LAST	11
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	-0.0200 ΔA
SAMPLE CYCLE	2	HIGH	0.0200 ΔA
VOLUME	3 μL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	30 μL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	300 μL	STD-2	
		STD-3	
CALCULATION		REPLICATE	DUPL
SAMPLE LIMIT	NO	DEVIATION	10%
REACTION DIRECTION	INCREASE	CONTROL	
CHECK	ON	CS1 POS	*
CONVERSION FACTOR	1.000	CS2 POS	*
OFFSET	0.000	CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 U/L
MEASUREMENT MODE	ABSORB	HIGH	1000.0 U/L
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	KINETIC
WAVELENGTH	340 nm	READING FIRST	5
DECIMAL POSITION	1	LAST	13
UNIT	U/L	REACTION LIMIT	0.2500 ΔA
		POINT	1
ANALYSIS		CALIBRATION	
POST DIL. FACTOR	2.00	CALIB. INTERVAL	ON REQUEST
CONC. FACTOR	NO	TIME	NO
SAMPLE CYCLE	1	BLANK RANGE LOW	-0.1000 A
VOLUME	8.0 μL	HIGH	0.4000 A
DILUTION NAME	H2O	REAGENT RANGE LOW	-0.0030 ΔA
VOLUME	10.0 μL	HIGH	0.0030 ΔA
REAGENT CYCLE	1	FACTOR	6878
VOLUME	190 μL	STANDARD POS	*
CALCULATION		CONTROL	
SAMPLE LIMIT	NO	CS1 POS	*
REACTION DIRECTION	INCREASE	CS2 POS	*
CHECK	ON	CS3 POS	*
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		TEST RANGE LOW	NO
MEASUREMENT MODE	ABSORB	HIGH	600 U/L
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	KINETIC
WAVELENGTH	340 nm	READING FIRST	5
DECIMAL POSITION	1	LAST	18
UNIT	U/L	REACTION LIMIT	0.3500 ΔA
		POINT	1
		CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2.00	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	-0.1000 A
SAMPLE CYCLE	1	HIGH	0.4000 A
VOLUME	8.0 μL	REAGENT RANGE LOW	-0.0030 ΔA
DILUTION NAME	H2O	HIGH	0.0030 ΔA
VOLUME	10.0 μL	FACTOR	13757
REAGENT CYCLE	1	CONTROL	
VOLUME	190 μL	CS1 POS	*
CALCULATION		CS2 POS	*
SAMPLE LIMIT	NO	CS3 POS	*
REACTION DIRECTION	INCREASE		
CHECK	ON		
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		TEST RANGE LOW	70.0 mEq /L
MEASUREMENT MODE	ABSORB	HIGH	150.0 mEq/L
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	CB
DECIMAL POSITION	1	LAST	10
UNIT	mEq/L	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2.00	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	2.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	20.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	400 µL	STD-2	NO
CALCULATION		STD-3	NO
SAMPLE LIMIT	0.1000 ΔA	REPLICATE	DUPL
POINT	T1	DEVIATION	10%
REACTION DIRECTION	INCREASE	CONTROL	
CHECK	ON	CS1 POS	*
CONVERSION FACTOR	1.000	CS2 POS	*
OFFSET	0.000	CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	800.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	1
DECIMAL POSITION	1	LAST	12
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2.00	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	2	HIGH	NO
VOLUME	3.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	30.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	300 µL	STD-2	NO
CALCULATION		STD-3	NO
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10%
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	12.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	0.0 mg/dL
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	2
DECIMAL POSITION	2	LAST	4
UNIT	mg/dL	REACTION LIMIT	*
<b>CALIBRATION</b>			
<b>ANALYSIS</b>		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2.00	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	20.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	20.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	200 µL	STD-2	NO
<b>CALCULATION</b>		STD-3	NO
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10%
CHECK	ON	<b>CONTROL</b>	
CONVERS FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 U/L
MEASUREMENT MODE	ABSORB	HIGH	2000.0 U/L
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	KINETIC
WAVELENGTH	340 nm	READING FIRST	3
DECIMAL POSITION	1	LAST	11
UNIT	U/L	REACTION LIMIT	0.2000 ΔA
		POINT	T1
CALIBRATION			
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2.00	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	0.3000 A
SAMPLE CYCLE	1	HIGH	1.0000 A
VOLUME	5.0 μL	REAGENT RANGE LOW	-0.2000 ΔA
DILUTION NAME	H2O	HIGH	0.2000 ΔA
VOLUME	20.0μL	FACTOR:	13227.0
REAGENT CYCLE	1	CONTROL	
VOLUME	225 μL	CS1 POS	*
CALCULATION		CS2 POS	*
SAMPLE LIMIT	0.2000 ΔA	CS3 POS	*
POINT:	T1		
REACTION DIRECTION	DECREASE		
CHECK	ON		
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		CONVERSION FACTOR	
MEASUREMENT MODE	ABSORB	OFFSET	1.000
REACTION MODE	R-S-SR1	TEST RANGE LOW	0.0 µg/dL
CALIBRATION MODE	SLOPE AVG	HIGH	500.0 µg/dL
REAGENT BLANK	REAG/DIL	NORM RANGE LOW	NO
CLEANER	NO	HIGH	NO
WAVELENGTH	550 nm	NUMBER OF STEPS	1
DECIMAL POSITION	1	CALCULATION STEPS A	ENDPOINT
UNIT	µg/dL	READING FIRST	3
ANALYSIS		LAST	10
POST DIL. FACTOR	NO	CALIBRATION	
CONC. FACTOR	NO	CALIB. INTERVAL	ON REQUEST
SAMPLE CYCLE	1	TIME	**
VOLUME	75.0 µL	BLANK RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	20.0 µL	REAGENT RANGE LOW	NO
REAGENT CYCLE	1	HIGH	NO
VOLUME	250 µL	STANDARD POS	*
START R1 CYCLE	4	STD-1	@
VOLUME	62.0 µL	STD-2	NO
DILUTION NAME	H2O	STD-3	NO
VOLUME	20.0 µL	REPLICATE	DUPL
CALCULATION		DEVIATION	10%
SAMPLE LIMIT	NO	CONTROL	
REACTION DIRECTION	INCREASE	CS1 POS	*
CHECK	ON	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	15.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	340 nm	READING FIRST	1
DECIMAL POSITION	2	LAST	6
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2.00	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	2	HIGH	NO
VOLUME	4.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	20.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	250 µL	STD-2	NO
CALCULATION		STD-3	NO
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 U/L
MEASUREMENT MODE	ABSORB	HIGH	700.0 U/L
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	KINETIC
WAVELENGTH	405 nm	READING FIRST	4
DECIMAL POSITION	1	LAST	12
UNIT	U/L	REACTION LIMIT	0.2500 ΔA
		POINT	T1
CALIBRATION			
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	5.00	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	-0.0500 A
SAMPLE CYCLE	1	HIGH	1.0000 A
VOLUME	5.0 μL	REAGENT RANGE LOW	-0.0100 ΔA
DILUTION NAME	H2O	HIGH	1.0100 ΔA
VOLUME	25.0 μL	FACTOR:	4594
CONTROL			
REAGENT CYCLE	1	CS1 POS	*
VOLUME	225 μL	CS2 POS	*
CALCULATION			
SAMPLE LIMIT	0.2000 ΔA	CS3 POS	*
POINT	T1		
REACTION DIRECTION	INCREASE		
CHECK	ON		
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		TEST RANGE LOW	0.0 mmol/L
MEASUREMENT MODE	ABSORB	HIGH	8.5 mmol/L
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	550 nm	READING FIRST	25
DECIMAL POSITION	2	LAST	37
UNIT	mmol/L	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	NO	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	15.0 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	300 µL	STD-2	NO
CALCULATION		STD-3	NO
SAMPLE LIMIT	ON	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 U/L
MEASUREMENT MODE	ABSORB	HIGH	250 U/L
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	FACTOR	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	KINETIC
WAVELENGTH	405 nm	READING FIRST	4
DECIMAL POSITION	1	LAST	8
UNIT	U/L	REACTION LIMIT	0.4300 ΔA
		POINT	T1
CALIBRATION			
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	5.00	TIME	**
CONC. FACTOR	NO	BLANK RANGE LOW	-0.0500 A
SAMPLE CYCLE	1	HIGH	1.0000 A
VOLUME	25 μL	REAGENT RANGE LOW	-0.0100 ΔA
DILUTION NAME	H2O	HIGH	1.0100 ΔA
VOLUME	25 μL	FACTOR:	1929
REAGENT CYCLE	1	CONTROL	
VOLUME	225 μL	CS1 POS	*
		CS2 POS	*
		CS3 POS	*
CALCULATION			
SAMPLE LIMIT	NO		
REACTION DIRECTION	INCREASE		
CHECK	ON		
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	400 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	1
DECIMAL POSITION	1	LAST	20
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	2,5 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	250 µL	STD-2	
CALCULATION		STD-3	
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	150.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	CB
DECIMAL POSITION	1	LAST	18
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2.00	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	20 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	30.0 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	200 µL	STD-2	NO
CALCULATION		STD-3	NO
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10%
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	NO
MEASUREMENT MODE	ABSORB	HIGH	180
REACTION MODE	R-S-SR1	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	600 nm	READING FIRST	11
DECIMAL POSITION	0	LAST	24
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	NO	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	2.5 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	20 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	190 µL	STD-2	
START R1 CYCLE	12	STD-3	
VOLUME	65	REPLICATE	DUPL
DILUTION NAME	H2O	DEVIATION	10.0 %
VOLUME	10		
CALCULATION		CONTROL	
SAMPLE LIMIT	NO	CS1 POS	*
REACTION DIRECTION	INCREASE	CS2 POS	*
CHECK	ON	CS3 POS	*
CONVERSION FACTOR	1.000		
OFFSET	0.000		

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GENERAL		OFFSET	
MEASUREMENT MODE	ABSORB	TEST RANGE LOW	0.000
REACTION MODE	R-S-SR1	HIGH	0.0 mg/dL
CALIBRATION MODE	SLOPE AVG	NORM RANGE LOW	400 mg/dL
REAGENT BLANK	REAG/DIL	HIGH	NO
CLEANER	NO	NUMBER OF STEPS	1
WAVELENGTH	600 nm	CALCULATION STEPS A	ENDPOINT
DECIMAL POSITION	1	READING FIRST	11
UNIT	mg/dL	LAST	24
ANALYSIS		CALIBRATION	
POST DIL. FACTOR	NO	CALIB. INTERVAL	ON REQUEST
CONC. FACTOR	NO	TIME	NO
SAMPLE CYCLE	1	BLANK RANGE LOW	-0.0700 A
VOLUME	3.0 µL	HIGH	0.8000 A
DILUTION NAME	H2O	REAGENT RANGE LOW	-0.0100 ΔA
VOLUME	20 µL	HIGH	0.0200 Δ A
REAGENT CYCLE	1	STANDARD POS	*
VOLUME	270 µL	STD-1	@
START R1 CYCLE	12	STD-2	NO
VOLUME	90 µL	STD-3	NO
DILUTION NAME	H2O	REPLICATE	DUPL
VOLUME	5 µL	DEVIATION	10%
POST DIL. FACTOR	NO		
CALCULATION		CONTROL	
SAMPLE LIMIT	NO	CS1 POS	*
REACTION DIRECTION	INCREASE	CS2 POS	*
CHECK	OFF	CS3 POS	*
CONVERSION FACTOR	1.000		

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	120.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	550 nm	READING FIRST	1
DECIMAL POSITION	1	LAST	12
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	3 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	300 µL	STD-2	
CALCULATION		STD-3	
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	4.5 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	CB
DECIMAL POSITION	2	LAST	6
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	3 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	10 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	300 µL	STD-2	
CALCULATION		STD-3	
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 g/Dl
MEASUREMENT MODE	ABSORB	HIGH	12 g/Dl
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	550 nm	READING FIRST	T1
DECIMAL POSITION	2	LAST	18
UNIT	g/dl	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	2	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	2	HIGH	NO
VOLUME	3 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	30 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	250 µL	STD-2	
CALCULATION		STD-3	
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK		ON	CONTROL
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	
MEASUREMENT MODE	ABSORB	HIGH	30 mg/L
REACTION MODE	R-S	NORM RANGE LOW	3000 mg/L
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	NO BLANK	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	600 nm	READING FIRST	1
DECIMAL POSITION	1	LAST	12
UNIT	mg/L		
ANALYSIS		CALIBRATION	
POST DIL. FACTOR	NO	CALIB. INTERVAL	ON REQUEST
CONC. FACTOR	3.00	TIME	NO
SAMPLE CYCLE	2	BLANK RANGE LOW	NO
VOLUME	5 µL	HIGH	NO
DILUTION NAME	H2O	REAGENT RANGE LOW	NO
VOLUME	10 µL	HIGH	NO
REAGENT CYCLE	1	STANDARD POS	*
VOLUME	300 µL	STD-1	@
		STD-2	
		STD-3	
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	ON		
CONVERSION FACTOR	1.000	CONTROL	
OFFSET	0.000	CS1 POS	*
		CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	800.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	BEFORE	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	500 nm	READING FIRST	1
DECIMAL POSITION	1	LAST	18
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	3	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	-0.0200
SAMPLE CYCLE	2	HIGH	0.1800
VOLUME	3,0 µL	REAGENT RANGE LOW	-0,0500
DILUTION NAME	H2O	HIGH	0.1800
VOLUME	10 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	250 µL	STD-2	
CALCULATION		STD-3	
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	INCREASE	DEVIATION	10.0 %
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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GENERAL		TEST RANGE LOW	0.0 mg/dL
MEASUREMENT MODE	ABSORB	HIGH	250.0 mg/dL
REACTION MODE	R-S	NORM RANGE LOW	NO
CALIBRATION MODE	SLOPE AVG	HIGH	NO
REAGENT BLANK	REAG/DIL	NUMBER OF STEPS	1
CLEANER	NO	CALCULATION STEPS A	ENDPOINT
WAVELENGTH	340 nm	READING FIRST	2
DECIMAL POSITION	1	LAST	8
UNIT	mg/dL	CALIBRATION	
ANALYSIS		CALIB. INTERVAL	ON REQUEST
POST DIL. FACTOR	3	TIME	NO
CONC. FACTOR	NO	BLANK RANGE LOW	NO
SAMPLE CYCLE	1	HIGH	NO
VOLUME	3 µL	REAGENT RANGE LOW	NO
DILUTION NAME	H2O	HIGH	NO
VOLUME	30 µL	STANDARD POS	*
REAGENT CYCLE	1	STD-1	@
VOLUME	220 µL	STD-2	
CALCULATION		STD-3	
SAMPLE LIMIT	NO	REPLICATE	DUPL
REACTION DIRECTION	DECREASE	DEVIATION	10.0 %
CHECK	ON	CONTROL	
CONVERSION FACTOR	1.000	CS1 POS	*
OFFSET	0.000	CS2 POS	*
		CS3 POS	*

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